


Measurements are taken from the ankle towards the thigh.

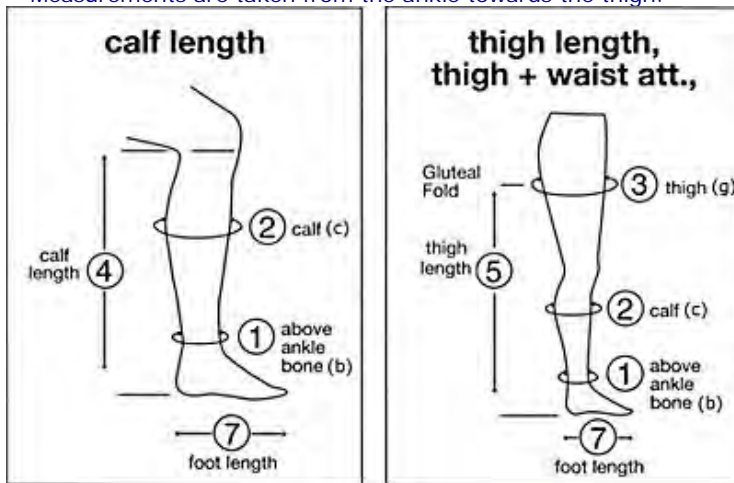

Measurement (b): This is the small area just above the ankle bone. It **determines the stocking size**.

Measurement checks:

Below knee Thigh length

b=ankle b=ankle

c=calf c=calf

g=thigh

As a general rule, the circumference of both legs should be measured so that any discrepancies between the two legs can be taken into consideration.

Size:

medi-Stockings are available in seven circumference sizes (I - VII). With these it is possible to provide 90% of all patients with standard-sized stockings, with the remainder being offered the facility of a custom-made stocking (called made-to-measure).